

ProOne Bitewing Parameters

1. Setup Imaging Software for the exposure.
2. Select the **Pan Bitewing** program on the GUI.
3. Select the **Patient Size** and **Jaw Size\Shape** on the GUI.
4. Press the **Return arrow** and have the patient stand as tall as possible while holding onto the patient handles.
5. Adjust the chin rest until it is the patient's collar bones area.
6. Have the patient stand still and as close to the unit as possible.
7. Raise the chin rest to the patient's chin and have the patient bite down on the bite stick.
8. Ask the patient to smile and set the occlusal plane parallel to the floor.
9. Align the midsagittal light directly between the patient's eyes and close the temple supports.
10. Align the layer light to the middle of the lateral incisor at the gum-line by pressing the left\right arrows on the GUI
11. Have the patient close their lips; press their tongue to the roof of their mouth and to breathe through their nose.
12. Press and hold the exposure switch when the unit is ready until no more audible noises are heard.

Patient	kV	mA
Child	70	4
Small Adult	70	5
Average-sized Male	70	6
Large Adult	70	7

For the best results, please have the patient remove all jewelry, dentures, medical devices, etc before taking the exposure.