

ProOne Pan Parameters

1. Setup Imaging Software for the exposure.
2. Select the **Pan Standard** program on the GUI.
3. Select the **Patient Size** and **Jaw Size\Shape** on the GUI.
4. Press the **Return arrow** and have the patient stand as tall as possible while holding onto the patient handles.
5. Adjust the chin rest until it is the patient's collar bones area.
6. Have the patient stand still and as close to the unit as possible.
7. Raise the chin rest to the patient's chin and have the patient bite down on the bite stick.
8. Ask the patient to smile and set the occlusal plane slightly below parallel to the floor.
9. Close the temple supports and align the midsagittal light directly between the patient's eyes.
10. Align the layer light between the lateral incisor and cuspid by pressing the left\right arrows on the GUI.
11. Have the patient close their lips; press their tongue to the roof of their mouth and to breathe through their nose.
12. Press and hold the exposure switch when the unit is ready until no more audible noises are heard.

Patient	kV	mA
Child	64	7
Small Adult	66	7
Average-sized Male	68	7
Large Adult	70	7

For the best results, please have the patient remove all jewelry, dentures, medical devices, etc before taking the exposure.